EyeGiene® Insta-Warmth™ System & Refills

Daily Treatment for Dry Eyes

ASK YOUR EYECARE SPECIALIST IF EYEGIENE IS RIGHT FOR YOU

Uses: The EyeGiene Insta-Warmth System & Refills provides therapeutic warmth for treating evaporative dry eye and chalazia.

⚠ Important Safety and Use Information: PLEASE READ CAREFULLY THE INSTRUCTIONS FOR USE AND ALL WARNINGS BEFORE USING THIS PRODUCT. TO REDUCE THE RISK OF BURNS AND PERSONAL INJURY, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE USE INSTRUCTIONS AND WARNINGS AND PRECAUTIONS.

WARNINGS

•**Contact Lens/Eye Surgery** - Remove contact lenses or other devices prior to use. If you had recent cataract or other eye surgery, ask your eyecare specialist before use.

• If you experience irritation, any discomfort or burning remove the EyeGiene Insta-Warmth System immediately.

⚠ For all consumers:

- Avoid additional pressure on the eye mask once it is in use.
- System is to be used with eyelids closed.
- Use only one (1) warming wafer at a time in each pouch of the eye mask.
- Following removal from the foil package, place the warming wafers in openings at the top of each side of the eye mask.
- Do not use on unhealthy, numb, damaged, or broken skin.
- Do not use after eyelid surgery or eye surgery without approval by your eye surgeon.
- Stop use if product feels too hot or is uncomfortable.
- Do not microwave or attempt to reheat. Warming units are for one-time use only. Discard appropriately after use.
- Do not use if eyelids do not close normally.
- Do not share the eye mask.

Additional important considerations:

- If you are concerned, consult your doctor as he/she is most familiar with your health.

Directions:

1.Prior to opening the warming wafers foil pouch, adjust the strap so eye mask fits snugly and comfortably against your eyelids.
2.Open the foil pouch, remove the two warming wafers, and insert one into the right side of the eye mask, and the other one into the left side of the eye mask.
3.Find a comfortable position and relax. Place the eye mask over both eyes with eyelids gently closed. For maximum effectiveness, we suggest that you wear the eye mask for at least 10 minutes.
4.When finished, discard the warming wafers. The eye mask can be used multiple times, and is washable by hand.

⚠ If you experience any discomfort, burning, swelling, rash, or other changes on your skin where the eye mask was worn; **STOP AND CALL YOUR DOCTOR IMMEDIATELY. Keep out of reach of children and pets.**

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